


How to Use the Topi-CLICK® Perl™ Vaginal Dosing Kit



A. Opening/Attaching the Snap-on Applicator to the Perl

1. Flip Perl shell cap open using integrated tab. (Figure a.)
2. Snap the Snap-on Applicator to the extended receptor port on the Perl shell. (Figure b.) **Note:** The Snap-on Applicator Plunger should be fully inside the applicator barrel when dosing as shown. (Figure b and c.)

Snap-on Applicator™

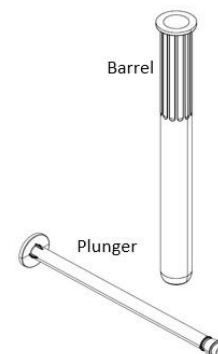

B. Loading Perl / Dispensing Medication

1. To dispense medication, simply turn/CLICK the base of the Perl to load the metered dose – as with all Topi-CLICKs, wait 15 to 30 seconds for full dose to load into the Snap-on Applicator. (Figure c.)

*** It is important to keep the Perl in vertical position when loading the Snap-on Applicator™ as shown. (Figure c.)**

Dosing Notes:

Each turn/CLICK dispenses 0.25 mL of medication accurately and does not need fill lines on the Snap-on Applicator. (see table below)

Dosing Guide	
1 CLICK	= 0.25 mL
2 CLICKs	= 0.50 mL
3 CLICKs	= 0.75 mL
4 CLICKs	= 1.00 mL


No Reference Lines on Applicator:

The Topi-CLICK is the most accurate** topical dispenser available. Unlike the traditional tube and applicator, the Snap-on Applicator was designed to be loaded by Topi-CLICK technology and therefore does not need reference lines to ensure accuracy.


** Go to www.doselogix.com/accuracy_study to request a free copy of the independent accuracy study.




"Flip Open" to Open

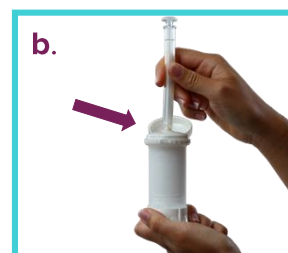

"Snap-On" to Load

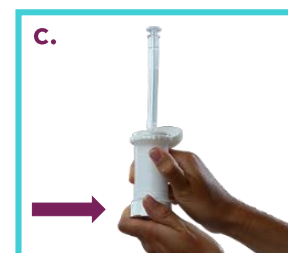

"Turn / CLICK" to Dispense

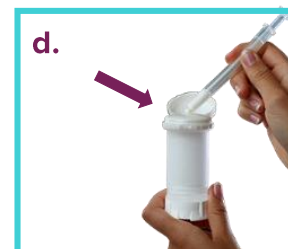

"Tilt-Off" to Remove

C. Removing the Snap-on Applicator

1. Simply tilt the Snap-on Applicator over as shown in (Figure d.) until the applicator releases from the receptor extended port on the shell.
2. **Reminder:** Depending on the patients removal technique, there may be a small dollop of cream remaining on the Snap-on Port which is part of the patient's dose.



- ◆ USP Class VI compliant
- ◆ BPA/Latex Free
- ◆ Hand wash applicator with warm water only; not dishwasher safe.

Perl Instructions Rev. 4/2016