

Topi-CLICK<sup>®</sup> 35  
**DOSE** 

Topi-CLICK<sup>®</sup> 35 DOSE <sup>™</sup>  
Topical Dosing Dispenser

Healthcare providers and caregivers can easily verify a dose with the NEW graduated Topi-CLICK<sup>®</sup> 35 DoseCheck<sup>™</sup> system.



*Provides an independent check system to verify dose.*



Features graduated marks at each 0.5 mL.

BOTTOM of Plunger indicates the amount of medication remaining.

Each turn/CLICK of base dispenses 0.25 mL of topical  
1 CLICK = 0.25 mL  
2 CLICKs = 0.50 mL

The Topi-CLICK<sup>®</sup> 35 DOSE<sup>™</sup> topical dosing dispenser provides the same advanced technology as the Topi-CLICK<sup>®</sup> 35 and offers EXTRA assurance with graduated marks so the dose starting and stopping points can be easily verified and recorded. ***Topi-CLICK<sup>®</sup> 35 DoseCheck<sup>™</sup> initial start point should begin when the bottom of the plunger is at the 30 mL graduated mark.***

**Consistent & Accurate Dose<sup>™</sup>**

- Topi-CLICK<sup>®</sup> dispensers proven most accurate among others tested by \*ARL
- Eliminates user-variability
- Consistent automated dosing throughout treatment
- Allows for easy recording of numerical volumes dispensed

**Built-in Applicator Pad**

- Reduces the risk of cross-contamination to others

**CLICK-Assist<sup>™</sup> Technology**

- Clear Audible CLICKs
- Built-in Visual CLICK points
- Strong Tactile Sensations at each CLICK point

**Viscosity Assurance<sup>™</sup>**

- Accuracy NOT affected by varying formulations
- Accuracy NOT affected by varying bases

Made of FDA grade materials | BPA and Phthalate free | UV block and is Recyclable 

Request  
\*ARL PEER REVIEWED EVIDENCE at  
[http://doselogix.com/accuracy\\_study](http://doselogix.com/accuracy_study).

## Topi-CLICK® 35 DOSE ™

### Healthcare Provider Use Instructions



Provides an independent check system to verify dose.

#### STEP 1:

Healthcare provider should begin by reading dose volume while holding the dispenser at eye level.

Record volume by reading the graduated line at the **bottom of the plunger** and then record by using the healthcare facility's preferred method of dose recording. (Figure a.)

**Holding dispenser above or below eye level may cause errors in reading. Topi-CLICK® 35 DoseCheck™ initial start point should begin when the bottom of the plunger is at the 30 mL graduated mark.**




**STEP 2:**  
Lift off the cap of the Topi-CLICK® dispenser by placing your thumb under the tab on the protective outer cap and pressing upward.



**STEP 3:**  
Watch the visual indicators when rotating base. Be sure to leave the base in the registered CLICK position by aligning the base column with the tube node after each use.



**STEP 4:**  
Turn the base to dispense the number of CLICKs or mLs specified in the instructions on the prescription label.

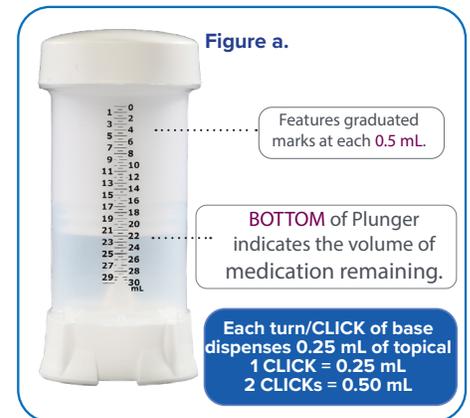


**STEP 5:**  
Apply topical medication as directed by the physician or pharmacist. Rub the area of the skin using the **built-in applicator pad**. Continue rubbing until topical is completely absorbed into the skin. **IMPORTANT:** The built-in applicator lessens the risk of cross-contamination and ensures the full dose is applied to intended area.



**STEP 6:**  
Replace cap after each dose to prevent evaporation. Be sure cap snaps on tightly.



**STEP 7:**  
Read volume stopping point by holding dispenser at eye level and recording as indicated by your healthcare facility. (Special Note: See Figure b.)

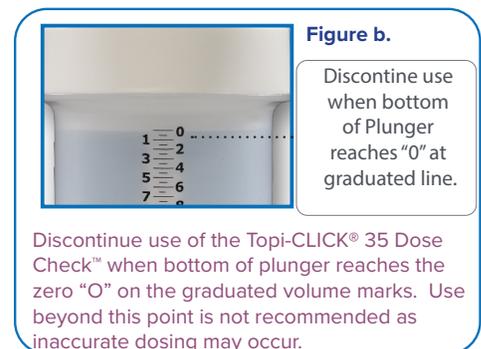

Follow instructions on prescription label before using the Topi-CLICK.®