


Topi-CLICK MICRO[®]



Micro Quick-Fill Technology[™]
Makes Filling Fast & Simple

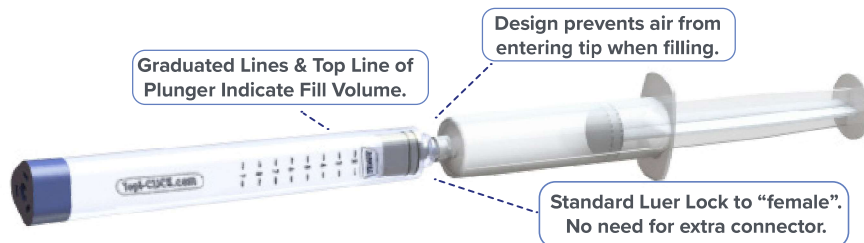

Note: Graduated Lines are for filling only and not for patients. Top black line of the plunger indicates Fill Volume. The embossed Refill Badge helps patients reorder medication in a timely manner.



Select one of 3 applicators

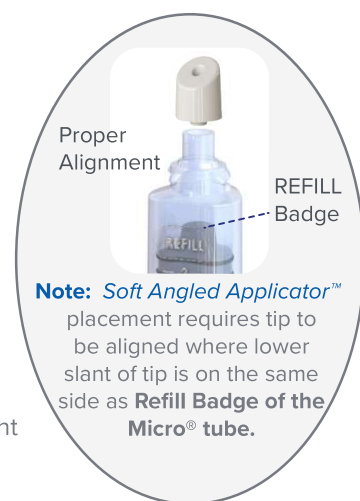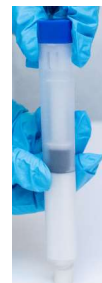

Step 1: Filling
Syringe Tip-to-Tip Filling-
Screw Luer syringe to the Luer lock tip of the Micro[®]. Requires no extra Luer Lock adaptor.

EMP Jar Quick-fill Adaptor[™] Filling- Screw on Topi-CLICK Micro[®] EMP Jar Quick-fill Adaptor[™]. Fill topical medications directly from mixing jar with this added Micro[®] accessory.

Step 2: Applicator Tip Attachment - Select Applicator Tip and press on by pushing straight down (important) against a hard surface.

Note: Soft Angled Applicator[™] placement requires tip to be aligned where lower slant of tip is on the same side as Refill Badge of the Micro[®] tube.




Step 3: Priming the Micro[®] Advance the actuator to engage plunger by manually turning/CLICKing base. The Actuator Bar should seat completely to the plunger before priming. Actuator can also be advanced by using a mini cordless screwdriver to speed up the process (200 RPM Max). A Micro Driver Socket Bit[™] can be purchased for faster actuator advancement with a cordless mini screwdriver. (See Advancing Actuator Instructions.)



Step 4: Cap Placement Screw on cap securely to protect tip and prevent evaporation.

Step 5: Label Application Apply above the base opposite the "REFILL Badge" indicator.

Step 3a - Priming

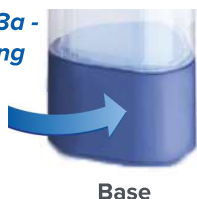

Step 3a: Priming Prime the Micro[®] by turning/CLICKing the base until contents are dispensed, then wipe off. Be sure to align Micro[®] base with the triad tube. Approximate Priming: 0.15 mL or 3 CLICKS.

Topi-CLICK Micro[®] 9 mL
Dispense/CLICK: 0.05 mL
Dispenser Volume: Up to 9.75 mL
Total Delivery Volume: Up to 9.45 mL
Approx. Residual: 0.15 mL
Approx. Priming: 0.15 or 3 CLICKS

Topi-CLICK Micro[®] 5 mL
Dispense/CLICK: 0.05 mL
Dispenser Volume: Up to 5.5 mL
Total Delivery Volume: Up to 5.35 mL
Approx. Residual: 0.15 mL
Approx. Priming: 0.15 or 3 CLICKS