


# Topi-CLICK MICRO<sup>®</sup> User Instructions



- ✓ Automated, metered topical dosing provides an accurate and consistent dose every time
- ✓ Audible, Tactile and Visual CLICKs indicate measure of dose (no need to manually mark dose)
- ✓ Innovative triad-shape allows for precision handling
- ✓ Built-in REFILL Badge helps patients reorder medication on-time



**STEP 1:**  
Unscrew the cap of the Topi-CLICK Micro<sup>®</sup> and set aside for recapping after use.

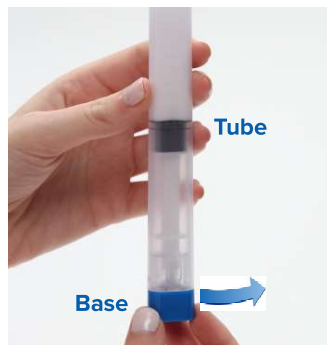

**STEP 2:**  
Holding the Micro<sup>®</sup> in an upright position, turn/CLICK the **Base** the number of CLICKs or mLs specified on the prescription label. Use the Audible, Tactile and Visual CLICK points of the triad-shaped **Base** as dosing indicators. Be sure to align **Base** with the **Triad Tube** to ensure dose is fully dispensed.



**STEP 3:**  
Apply topical medication with built-in applicator as directed by physician or pharmacist. **Continue rubbing until preparation is completely absorbed into the skin.**  
**IMPORTANT:** The built-in applicator lessens the risk of transference to unintended areas. No need to use your fingertips to apply medication.



**STEP 4:**  
Screw on cap after each dose to prevent evaporation. Be sure cap is secured completely.

Innovative TRIAD Shape Allows Precision Application Control

Each turn/Click of the base dispenses 0.05 mL of medication accurately.



REFILL Line  
Plunger

## The Refill Badge™ reminder system helps patients reorder in a timely manner.

When the **TOP** black line of the **Plunger** reaches the embossed line below the word "REFILL" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "REFILL".

*Note: Graduated Marks are for pharmacy filling purposes only. Follow instructions on prescription label before use. Topi-CLICK Micro<sup>®</sup> is not re-fillable.*

Made of FDA grade materials | Phthalate free | blocks UV and is Recyclable ♻️.