

Flying and Airport Security



Traveling with any of our Topi-CLICK® by DoseLogix devices fit within the TSA guidelines for medication in carry-on bags.

If your prescription has been filled in our Topi-CLICK® 35, PERL™ or Micro™ device, the cream or gel volume will be less than the 100 milliliter TSA guideline maximum for liquids, cream or gels.

If your prescription has been filled in our larger Topi-CLICK® 140 device, the volume can be up to 140 milliliters. This is above the TSA guideline maximum of 100 milliliters, however you may still bring your Topi-CLICK® 140 device in your carry-on bag based on the “3-1-1 Liquids Rule Exemption”. Please go to www.tsa.gov/travel/special-procedures for specifics and/or any updates to TSA guidelines.

To help avoid leakage due to cabin pressure changes, Topi-CLICK® by DoseLogix recommends the following:

1. Remove the cap of the Topi-CLICK® container.
2. Place a square of plastic wrap over the entire applicator (the smooth top with the hole or holes).
3. Replace the cap securely.

Additionally, place your Topi-CLICK® device in a sealed zip-top bag in your carry-on for ease of TSA screening and to avoid the possibility of having your prescription lost or stolen in checked luggage.