

## Traveling with Topi-CLICK® Dispensers and Airport Security



Traveling with any of our Topi-CLICK® devices fit within the TSA guidelines for medication in carry-on bags.

Prescriptions filled using our Topi-CLICK® 35, Topi-CLICK Perle® or Topi-CLICK Micro® dosing applicators, the volume of the cream or gel will be less than the 100 milliliter TSA guideline maximum for liquids, creams or gels.

Prescriptions filled using the Topi-CLICK® 140 dosing applicator, the volume of the cream or gel can be up to 140 milliliters. This is greater than the TSA maximum of 100 milliliters, however you may still pack your Topi-CLICK®140 applicator in your carry-on bag based on the “3-1-1 Liquids Rule Exemption”. For more details on the specifics and/or any updates to TSA guidelines visit [tsa.gov/travel/security-screening/liquids-rule](https://www.tsa.gov/travel/security-screening/liquids-rule) .

To prevent leakage due to cabin pressure changes, we recommend the following:

1. Remove the cap of the Topi-CLICK® device.
2. Place a square of plastic wrap over the entire applicator (the smooth top with the hole or holes).
3. Replace the cap securely.

Additionally, place your Topi-CLICK® device in a sealed zip-lock bag in your carry-on bag for ease of TSA screening and to prevent the possibility of having your prescription lost or stolen if packed in checked luggage.

