

## FILLING THE APPLICATOR



**Transfer** the desired amount of compounded cream into the UnoDose™ Applicator barrel using a spatula, ointment jar, or filling system.



For best results, remove air during filling. Start with a small amount to **fill the corners** in the bottom of the barrel. **Tap the bottom** of the applicator on your palm or a countertop to settle the cream and remove air.



**Add the remaining cream** in several steps while continuing to **tap the bottom** of the applicator to settle the cream and remove air. Leave the top of the cream flat and smooth.



Put the applicator cap on the barrel and **push downward to snap** into place. The applicator cap will butt up against the ring on the barrel. **Tap the bottom again** and confirm that no air remains in the cream.



**Prime** to remove air. For a fast prime, use the prime **tool supplied** in each box. **Push slowly** until cream just comes out the holes, or the tool stops. **NOTE:** The short end of the tool leaves 32 usable milliliters in the applicator, the long end leaves 17.




Finish priming by **turning the dosing knob** several clicks until cream exits all holes smoothly and equally. **NOTE:** Be sure to leave the knob in a “lock” position, and not between clicks.



**Wipe away** the cream from the applicator cap, then **lock the protective cover** in place with a light push and clockwise turn. You will hear and feel a click.



**Confirm** the correct volume on the scale by reading **at the blue line**. **NOTE:** The scale shows the usable volume and clicks remaining, and excludes the residual volume of 1.5 milliliters.

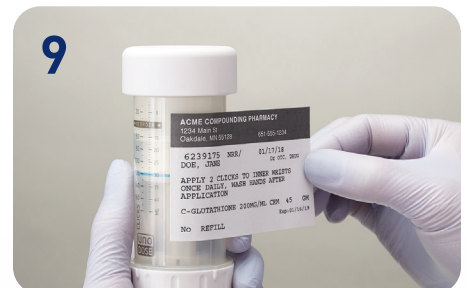

Apply a **prescription label** to the barrel. **NOTE:** Do not cover the CLICKS scale.

