

INSTRUCTIONS FOR USING THE UNODOSE™ APPLICATOR




1 Remove the protective cover with a light counter-clockwise twist and pull while holding the barrel.



2 Turn the dosing knob in the direction marked for the number of prescribed clicks.



NOTE: Be sure to leave the knob in a “lock” position, and not between click positions.

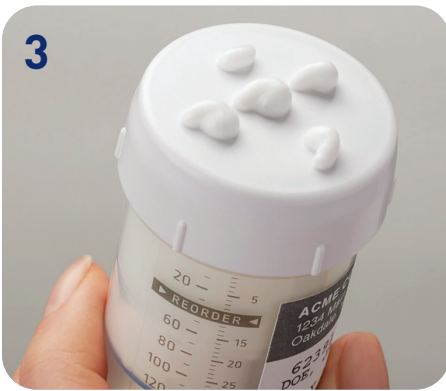

3 Confirm that cream has exited the holes of the applicator cap.



4 Apply the cream to the skin absorption site.

Rub with the applicator cap until the cream is evenly spread (per physician or pharmacist direction).

Replace the protective cover with a light push and clockwise turn. You will hear and feel a click.



5 Check the scale at the blue line for the number of clicks remaining.

Refill your prescription as instructed by your pharmacy.

PRECAUTIONS

- Store in a dry place at room temperature.
- Avoid exposure to sunlight and other bright lights.
- When traveling in an airplane or to a different elevation, seal the applicator in a zip lock bag to prevent soiling from any leakage due to pressure changes.



Topi-CLICK | TEAM Technologies
300 Parkbrooke Place, Suite 210, Woodstock, GA 30189
877-870-8448 | info.TopiCLICK@teamtech.com
www.Topi-CLICK.com



**Pharmaceutical
Delivery
Devices**