

INSTRUCTIONS FOR USING TOPI-CLICK® 35 CHILD RESISTANT

STEP 1: Unlock and Remove Child Resistant Cap



Read instructions on prescription label.

Remove the *Child-Resistant Cap. Place palm over the cap and press downward firmly while turning counterclockwise. Once unlocked, lift the cap off the dispenser.

Tip: The top of the Child-Resistant Cap includes the instructional text "PUSH DOWN & TURN" as well as directional arrows labeled "OPEN" and "CLOSE" as a convenient visual reminder of the proper technique for cap opening and closure.

STEP 4: Apply Medication Using Built-in Applicator



Apply topical medication by rubbing the area of skin as directed by your pharmacist using the **built-in applicator**. Continue rubbing until topical is completely absorbed into the skin.

IMPORTANT: The Built-in Applicator lessens the risk of cross-contamination and ensures the full dose is applied to the intended area.

STEP 2: Locate Two Safety Tabs-each labeled PUSH on DoseLock™ Base



Locate the two safety tabs - each labeled "PUSH"- on the base of dispenser.

Simultaneously press two DoseLock™ Safety Tabs (i.e., tabs labeled "PUSH") of base using grip with one hand while holding dispenser tube firmly in other hand.

STEP 3: Turn DoseLock™ Base to Dispense Medication



While maintaining pinch grip, rotate the DoseLock™ base to deliver the number of CLICKS specified in prescription.

STEP 5: Reinstall and Lock Child-Resistant Cap



Replace cap after each dose to prevent evaporation. Align the Child-Resistant Cap over the applicator and press down while turning clockwise until the cap is securely locked.

STEP 6: Monitor the Refill Indicator



The bottom edge of the plunger is the indicating line; when it reaches the embossed line below the word "Refill" approximately 32 CLICKS are left. Approximately 12 CLICKS remain at the line above the word "Refill".

*Certified Child Resistance. Meets U.S. Consumer Product Safety Commission (CPSC) criteria for child-resistance under 16 CFR §1700.20. (only when fully assembled as specified.)

Topi-CLICK® 35 is the most accurate topical dosing dispenser among those compared in an independent study titled Topical Metered Dosing Dispenser Performance Evaluation by Analytical Research Laboratories. Request study at https://topi-click.com/accuracy_study/.

PRECAUTIONS

- Store in a dry place at room temperature.
- Avoid exposure to sunlight and other bright lights.
- When traveling in an airplane or to a different elevation, seal the applicator in a zip lock bag to prevent soiling from any leakage due to pressure changes.



Topi-CLICK | TEAM Technologies
 300 Parkbrooke Place, Suite 210, Woodstock, GA 30189
 877-870-8448 | info.TopiCLICK@teamtech.com
www.Topi-CLICK.com



**Pharmaceutical
Delivery
Devices**