

## FILLING THE APPLICATOR



### STEP 1: Fill the Dispenser Tube

Fill the Topi-CLICK® 35 \*Child Resistant dispenser with the desired amount of topical formulations using a Topi-CLICK® RapidFill™ System, spatula, EMP jar, or nozzle.



### STEP 2: Settle the Formula (if needed)

If needed while filling, gently tap the base of the dispenser on your palm, counter, or a Tap-n-CLICK™ device to help settle the topical formula and remove air pockets.



Key Tabs

Receiving Slots

### STEP 3: Install the Child-Resistant Applicator

Align the internal key tabs of the applicator with the receiving slots at the top of the dispenser tube. Press down until the applicator snaps into place.



Locking Feature

**NOTE:** Only use the Child-Resistant Applicators that are supplied with the Topi-CLICK® 35 Child Resistant dispenser product. Always verify that the appropriate, child-resistant applicator is installed by confirming presence of the distinct, raised locking features located around the outer edge of the applicator.



DoseLock™  
Safety Tab

### STEP 4: Prime the Dispenser

Simultaneously press two DoseLock™ Safety Tabs (i.e., tabs labeled "PUSH") of base using pinch grip with one hand while holding dispenser tube firmly in other hand. While maintaining this pinch grip, slowly rotate the base until a small amount of medication is dispensed from the applicator.



Wipe excess topical from the applicator after priming is complete.

**NOTE:** Priming typically requires approximately 2 CLICKs (0.50 mL).



### STEP 5: Install the Child-Resistant Locking Cap

Place the outer cap on top of the dispenser. Push down firmly while turning clockwise to engage and lock the child-resistant mechanism.



**NOTE:** Verify the cap is securely locked by confirming it cannot be removed without using the push-down-and-turn action.

*\*Certified Child Resistance. Meets U.S. Consumer Product Safety Commission (CPSC) criteria for child-resistance under 16 CFR §1700.20. (only when fully assembled as specified.)*



### STEP 6: Apply the Label

Apply a prescription label to the tube.

**NOTE:** Do not cover the Refill badge.



**Topi-CLICK | TEAM Technologies**  
 300 Parkbrooke Place, Suite 210, Woodstock, GA 30189  
 877-870-8448 | [info.TopiCLICK@teamtech.com](mailto:info.TopiCLICK@teamtech.com)  
[www.Topi-CLICK.com](http://www.Topi-CLICK.com)



**Pharmaceutical  
Delivery  
Devices**